


# iSense Micro Receiver



- ① iSense Micro housing
- ② ON/OFF switch and battery compartment (312 battery)
- ③ Retention piece
- ④ Flex
- ⑤ Ear wax filter

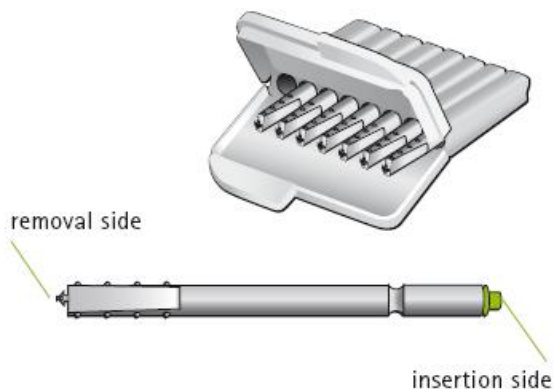

## Switching ON

Close the battery compartment by pressing on the base of the instrument.



## Switching OFF

Using the nail grip, slightly open the battery cover until it snaps into its OFF position.



## iSense Acoustical Warning Tones (Beep) Vocabulary

- Replace **Ear Wax Filter** = 3 **short** beeps every 5 seconds (beep\_beep\_beep...  
beep\_beep\_beep...)
- Replace **Battery** = 3 **long** beeps (beeeep\_beeeeeep\_beeeeeep)
- Ear Wax Monitoring = 1 **short** beep every hour